

**Product Name:** Self Sealing Sterilization Pouch

**Product Feature:** Raw material are medical dialysis paper and medical compound film.

The pouch with the function of bacterial barrier, easy to use, and will not being polluted after sterilized.

**Application:** Hospital, Clinic, Nail Beauty, Cosmetology and so on.

**Sterilization Method:** Steam, EO and FORM

**Instruction:**

1.Put the item into the pouch, peel off the strip from adhesive band, fold along dotted line and press.

2.Sterilization Condition:

2.1. Steam Sterilization:

Sterilized for 20-30min at degree of 121°C

Sterilized for 4min at degree of 132°C

Item volume should be between 5%-80% of sterilizer.

2.2. EO Sterilization:

when ethylene oxide concentration is 800~1000mg/L, relative temperature is 55°C~60°C, relative humidity is 60%~80%, sterilized for 6 hours.

3. After processed, if the sterilization indicator color changed (When EO processed, arrow turns yellow, when Steam sterilized, arrow turns brown), it shows sterilization finished.

4.Peel off the pouch from top and take out items.

**Notice:**

The pouch is single use product. Please destroy it after use.

1. After sterilized, open the pouch, it can be used.
2. Do not use if packing is damaged.
3. Do not use if paper layered.
4. The valid time is 6 months after sterilization. Please re-pack it and sterilize when more than 6 months.